


Instructions for the rapid **Status** COVID-19 Antigen Test For Home Use

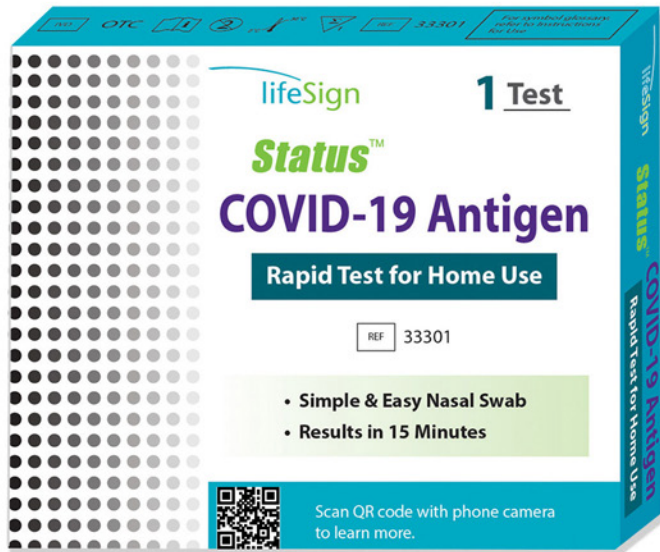

Have these items ready before starting a test *



1 Test in
foil pouch



1 Swab
in wrapper



1 Reagent
Capsule



Clock or
Timer

* Please read the complete package insert for details

STEP 1

Open the pouch
and remove the test



Place the test on
a flat surface



STEP 2

Twist the tab
to remove top




Squeeze the capsule to empty
all of the Reagent into the
Extraction Well



STEP 3



Remove the swab from the wrapper

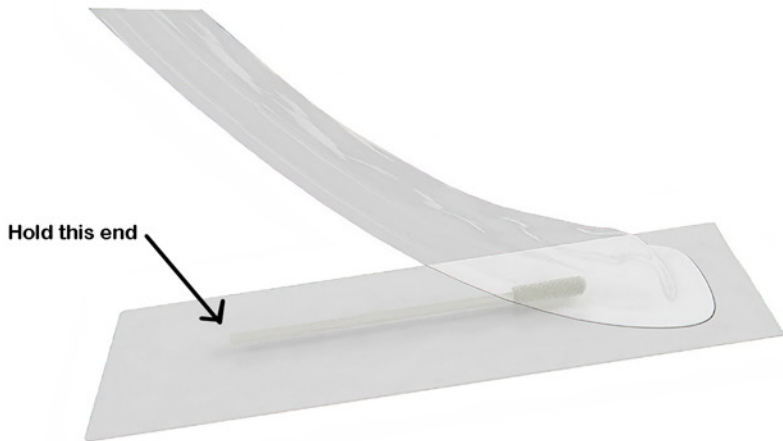

**OPEN THE SWAB WRAPPER AT THE STICK END. BE CAREFUL
NOT TO TOUCH THE FOAM TIP OF THE SWAB.**

STEP 4

Insert the swab into the nose and rotate the swab 5 times while pressing against the nostril. Using the same swab, repeat on the other side.

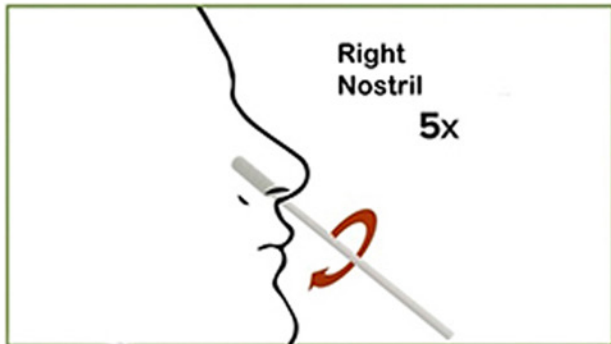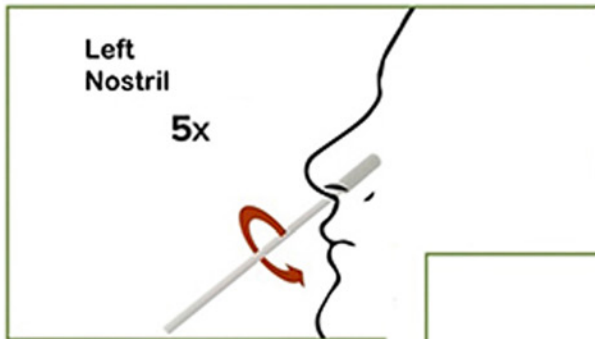

STEP 5

Insert the swab into the
swab stand
Rotate the swab 3x



Leave the swab in the device
and set a timer for 1 minute



STEP 6

After 1 minute, rotate the swab again 3x.
Remove swab and discard.



STEP 7

Gently raise the device upright and let stand for 3 seconds



Slowly lay the test back down and set a timer for 15 minutes



STEP 8

Read the Results



NEGATIVE
COVID-19



POSITIVE
COVID-19



INVALID

Repeat test
using a new device